

## BALMORAL ACTIVE MATTRESS



**Product Code:**

HAMBALM001

**Description:**

- The mattress is made up of 27 individual cells; 24 alternating air cells and 3 static pillow section cells
- The mattress has an integrated foam base
- Cells are made of pure PU
- Pillow section for patient stability
- All cells are interchangeable for ease of repair and cleaning
- 4-way stretch welded PU cover which is vapour permeable and water resistant and has a Crib 5 fire rating
- Compatible with Harvest Healthcare's pumps 2 and 3

**Mattress Dimensions:**

2000 x 880 x 150 mm

**Weight:**

8kg

**Max. Patient Weight:**

Harvest 2 Pump: 29 stone (184kg)  
Harvest 3 Pump: 40 stone (254kg)

**Pressure Relief Risk:**

Very High

**Pump Technical Specification:**

Please see separate pump specification sheet

**Cleaning Guidance:**

Cover should be wiped down with hot water and mild detergent. If heavily soiled, the cover may be machine washed up to 60-95°C and tumble dried. For disinfection use sodium hypochlorite solution max 1000ppm. Use no bleach, biological or phenolic-based cleaners. Dry completely before storing.

**Warranty**

In terms of Harvest Healthcare Guarantee and Warranties the Harvest Balmoral mattress is guaranteed for three years from date of purchase.

**Exclusions:**

- Damage to fabric covers through incorrect use and penetration by sharp instruments is excluded.



Special Products and Service

**Contact Us**

Units 5 and 6 Lower Vicarage Road  
Southampton  
Hampshire  
SO19 7RJ

T: 02380 420678

E: [info@easycare systems.co.uk](mailto:info@easycare systems.co.uk)

[www.easycare systems.co.uk](http://www.easycare systems.co.uk)