

QUALITY
SOLUTIONS
since
1998
ACCESS &
EVACUATION

rampCENTRE[®]

easy**care**
systems


by
Enable  Access

The Access & Ramp Assessment Guide 2016/17

helping you care



Units 5 to 6 Lower Vicarage Road, Southampton, Hampshire, SO19 7RJ
T: 02380 420678 E: info@easycare systems.co.uk W: www.easycare systems.co.uk

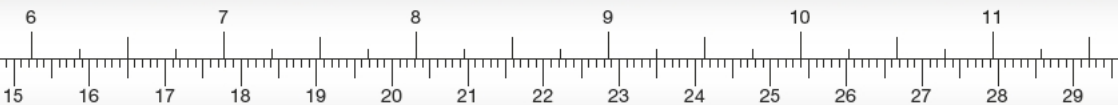

Get ramps right

with our famous ramp specifier guide

1 What type?

Steps/Stairs



Wood/Level Access Door Threshold



PVC Door Threshold



Vehicle Access



2 How steep?

1:6 (10°)*



Acceptable in some cases for short threshold ramps.

1:8 (8°)*



A useful balance between gradient and space suitable for portable ramps in some assisted applications and events etc.

1:12 (5°)



Often required by social services/local authorities. Minimum required Building Regs Doc M for installed ramps.

1:15 (4°)



Sometimes required by social services/local authorities for independent access.


* Wheelchairs must descend backwards. Assess safety/risk.

3 How long?

- a. Measure the step height / level difference.
(Total height of steps or height to resting point on vehicle).



Remember:
"The longer the ramp, the easier the slope".

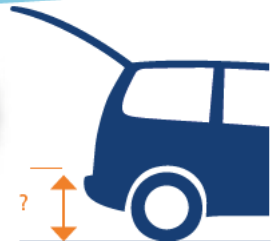

3 continued

b. Use the **Ramp Length Guide** (below) to decide which length you need.

Step height/level difference (up to)	Length at 1:6	Length at 1:8	Length at 1:12
10cm (4")	60cm (2')	80cm (2' 8")	120cm (4')
15cm (6")	90cm (3')	120cm (4')	180cm (6')
20cm (8")	120cm (4')	160cm (5' 4")	240cm (8')
25cm (10")	150cm (5')	200cm (6' 8")	300cm (10')
30cm (1')	180cm (6')	240cm (8')	360cm (12')
45cm (1' 6")	270cm (9')	360cm (12')	540cm (18')
60cm (2')	360cm (12')	480cm (16')	720cm (24')

Note: Ramp lengths over 300cm (9'10") can be achieved using high-rise ramp kits and Welcome® Ramp System.

4 Select product height & type



or select by product type...

Full Width Portable Ramps & Combination Kits

Great for all types of mobility device, including 3 wheel scooters. Easy to walk on, with full width grip surface

4

Channel Ramps

Great for narrow doorways. Separate halves allow very compact storage and lighter carrying

7

Threshold Ramps

Simple to deploy. Can be used for either one or both sides of a door

8

Relocatable Ramps

Great for semi-permanent access. Easy to re-deploy elsewhere

10

Modular Ramp & Half Step Systems

Quick to fit & easy to remove or modify when needs change. Perfect for independent access

12

Then turn to the product page and select the ramp you need.

aerolight[®] by rampCENTRE[®]

“Full width, high capacity, lightweight ramps”

Your benefits:

- ▶ use with all types of scooters and wheelchairs
- ▶ unique lip design gives incredible grip at each end and allows the ramp to be used either way around
- ▶ quick and easy deployment
- ▶ compact storage features



Designed & Manufactured in the UK.



aerolight-xtra™

“Proven & popular for building access”

Your benefits:

- ▶ folds for compact storage
- ▶ contrasting colour edge strip increases visibility
- ▶ many uses in public and domestic building access
- ▶ full width anti-slip provides excellent grip
- ▶ clips together when folded




Code	Length
* AX6	60cm (2ft)
* AX9	90cm (3ft)
* AX12	120cm (4ft)
* AX15	150cm (5ft)
* AX18	180cm (6ft)
AX21	210cm (7ft)
AX24	240cm (8ft)

aerolight-classic™

“Excellent value with great features”

Your benefits:

- ▶ a great portable solution for thresholds, kerbs and steps
- ▶ well suited to public access
- ▶ simple to deploy

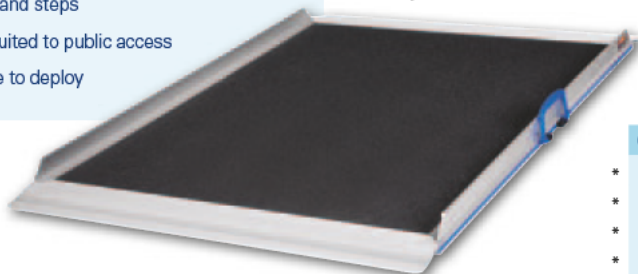

Code	Length
* AC6	60cm (2ft)
* AC9	90cm (3ft)
* AC12	120cm (4ft)
* AC15	150cm (5ft)



Your benefits:

- ▶ extremely simple to use
- ▶ great for 'out and about'
- ▶ use with all types of scooters and wheelchairs
- ▶ ramp simply splits in two weighing as little as 8kg easy carrying and deployment



aerolight-broadfold™

Quick & easy to deploy, great for vehicle access

NEW! NEW-ERA Aerolight



rollouttrackway

Cover grass, gravel and other uneven surfaces for access with this incredibly hard wearing product. Can be trimmed.



Code	Length	Width	Weight kg (lb)
RT15	150cm (4' 11")	90cm (2' 11")	12 (26)
RT25	250cm (8' 2")	90cm (2' 11")	20 (44)
RT50	500cm (16' 4")	90cm (2' 11")	40 (88)

Wall Strap Kit


Code: AWK



Quick-fit strap holds all aerolight ramps against wall vertically for tidy storage.

Connect Platform

For use with aerolight ramps, excellent for events.

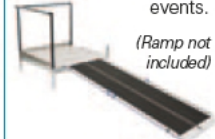

(Ramp not included)

CP12 Platform only
CPHK Platform handrail kit



350kg+



Code	Length
AB15	150cm (5ft)
AB18	180cm (6ft)
AB21	210cm (7ft)
AB24	240cm (8ft)

*

*

NEW

rampkit™ - high-rise


“Achieve lengths of up to 4.8m (16') with lightweight, portable products”

Your benefits:

- ▶ high steps can be overcome with these lightweight, portable ramp kits
- ▶ same fantastic features of the Aerolight-Xtra, just much longer lengths can be achieved
- ▶ ramp joining support also available separately - see below
- ▶ saves £'s by buying as a kit



350kg+*

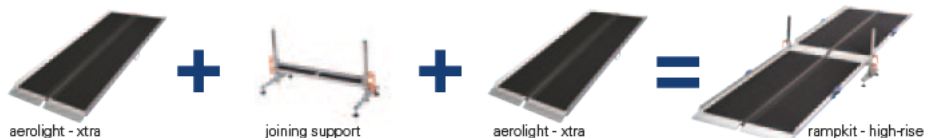



	Code	Includes	Total Length	Code	Description
*	HR30	1 x AJ4, 2 x AX15	300cm (10ft)	AJ4	Joining Support: Includes easy-use handle fixings for height adjustment. Can be used with any aerolight ramp.
*	HR36	1 x AJ4, 2 x AX18	360cm (12ft)		
	HR42	1 x AJ4, 2 x AX21	420cm (14ft)		
	HR48	1 x AJ4, 2 x AX24	480cm (16ft)		

rampkit™ - up & over

“The complete solution to PVC door thresholds”

Your benefits:

- ▶ quick and easy deployment
- ▶ copes with many threshold types and heights
- ▶ order as a complete kit to save any hassle
- ▶ portable and easy to store
- ▶ saves £'s by buying as a kit

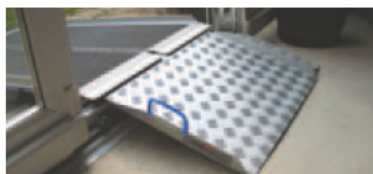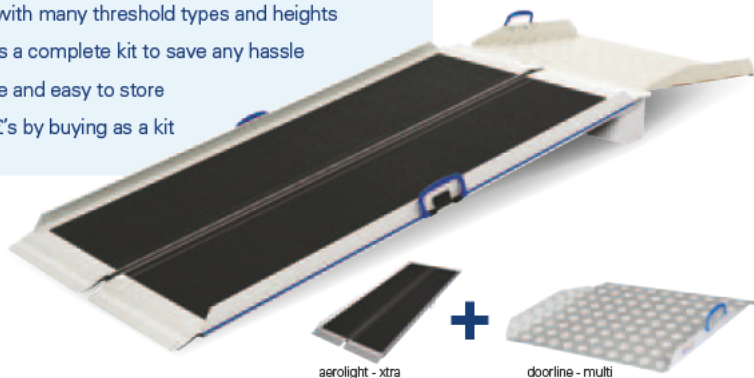

Code	Includes	External Ramp Length
UO12	1 x DM5, 1 x AX12	120cm (4ft)
UO15	1 x DM5, 1 x AX15	150cm (5ft)
UO18	1 x DM5, 1 x AX18	180cm (6ft)
UO21	1 x DM5, 1 x AX21	210cm (7ft)

ultralight™ by rampCENTRE®


Your benefits:

- ▶ low kerbs and long bevel sides greatly increase clearance for scooters & power chairs
- ▶ unique perforated surface and excellent grip design
- ▶ generous widths for safety and ease of use
- ▶ compact storage features



ultralight-rigid™

ultralight-folding™

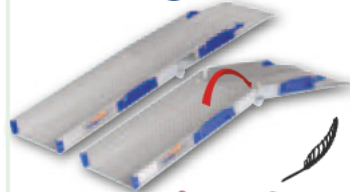

Code	Max Length
PF11	110cm (3'7")
PF15	150cm (4'11")
PF20	200cm (6'7")


Code	Length
PR06	55cm (1'10")

Storage bag also available
Code: PHB06



ultralight-telescopic™

Adjustable in use & compact to store

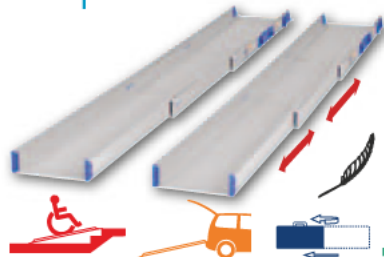

Code	Max Length
PT12-2	120cm (3' 11")
PT20-2	202cm (6' 8")
PT20-3	199cm (6' 6")
PT29-3	288cm (9' 5")


Extend to and use at any length

ultralight-combi™

Straight or up & over a threshold, even PVC



Combines both folding and telescopic features

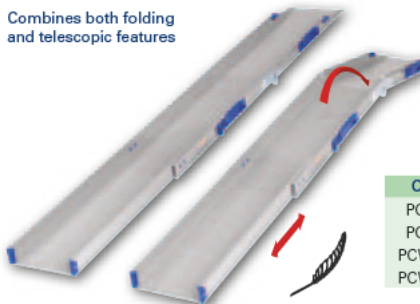

PCW; Extra wide options for larger mobility devices

Code	Max Length
PC21-3	207cm (6' 9")
PC28-3	276cm (9' 1")
PCW21-3	210cm (6' 11")
PCW28-3	300cm (9' 10")



doorline™ by rampCENTRE®


Your benefits:

- ▶ huge range of sizes to meet almost every threshold requirement
- ▶ custom ramps for the few applications our stock products cannot solve
- ▶ each product has an excellent grip surface



doorline-wedge™


“Simple threshold solution”

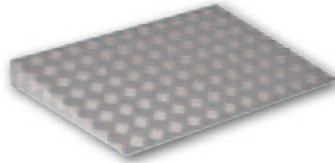

Code	Height
DW4	4cm (1.6ins)
DW6	6cm (2.4ins)
DW8	8cm (3.1ins)
DW10	10cm (3.9ins)

doorline-bridge™


“Bridge over almost any door threshold”

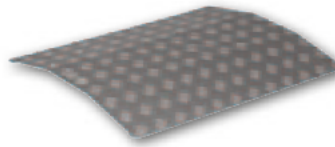

Code	Height
DB4	4cm (1.6ins)
DB6	6cm (2.4ins)
DB8	8cm (3.1ins)
DB10	10cm (3.9ins)

doorline-multi™


“Use with any aerolight ramp to overcome PVC doors”

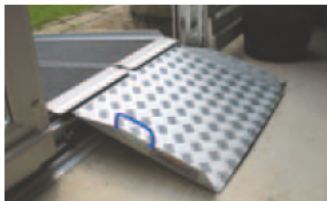

Code	Max Height
DM5	8cm (3.1ins)
DM7	12cm (4.7ins)

doorline-neatedge90™

“Single door width rubber wedge”

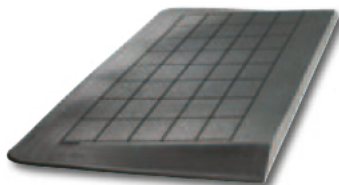


Code	Height
* DN15-9	1.5cm (0.6ins)
* DN25-9	2.5cm (1ins)
* DN50-9	5cm (2ins)
* DN75-9	7.5cm (3ins)

doorline-neatedge™

“Rubber threshold solution for wider doors”



up to 152cm (5') wide.

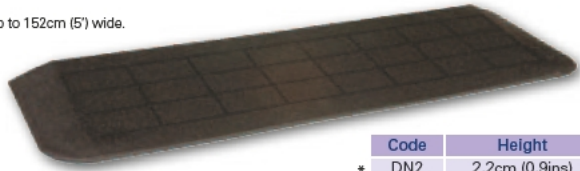

Code	Height
* DN2	2.2cm (0.9ins)
* DN3	3.2cm (1.3ins)
* DN4	4.5cm (1.8ins)
* DN5	5.1cm (2.0ins)
* DN6	5.7cm (2.3ins)



doorline-variwedge™

“World first concept adjustable wedge ramp!”



Assemble or adjust for re-use in less than a minute!

2 - 3cm

DVWS - Small

3.5 - 5.5cm

DVWM - Medium

6 - 10cm

DVWL - Large

3 sizes for up to 10cm (4")



Code	Min Height	Max Height
(SMALL) DVWS	2cm (0.8ins)	3cm (1.2ins)
(MEDIUM) DVWM	3.5cm (1.4ins)	5.5cm (2.2ins)
(LARGE) DVWL	6cm (2.4ins)	10cm (3.9ins)

doorline-neatslope™

“Durable & flexible, compact solution”



Code	Height	Self Adhesive Y/N
DNS6	0.6cm (0.25ins)	N
DNS6A	0.6cm (0.25ins)	Y
DNS10	1cm (0.4ins)	N
DNS10A	1cm (0.4ins)	Y
DNS16	1.6cm (0.6ins)	N
DNS16A	1.6cm (0.6ins)	Y

doorline-custom™

“Made to your specifications”



DCW

DCWL

DCBD



DCK




DCKL



DCKLL



DCLP

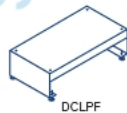

DCLPF



permaramp™ by rampCENTRE®

“Leave in place for all types of door threshold”


Your benefits:

- ▶ eliminates requirement for the ramps to be lifted once in position
- ▶ easily adjust and fix down or leave in place for every threshold type
- ▶ unique 'gripdeck' surface for sustained outside use
- ▶ fixing options at top and bottom for secure location

Designed & Manufactured in the UK.



permaramp-original™ “Excellent value & weight capacity”

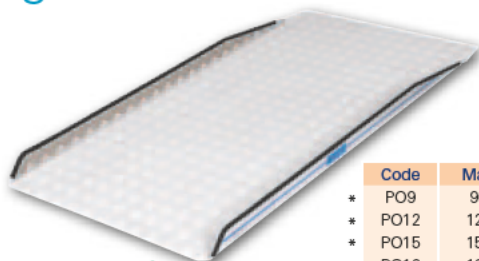


350kg +

Code	Max Length
* PO9	90cm (3ft)
* PO12	120cm (4ft)
* PO15	150cm (5ft)
* PO18	180cm (6ft)
* PO21	210cm (7ft)
* PO24	240cm (8ft)



permaramp-adjust™ “Self supporting, great for PVC doors”

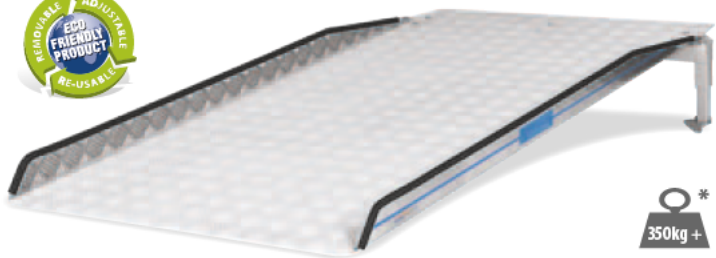

7.5cm (3") to 22.5cm (9")


350kg +



Code	Max Length
* PA9	90cm (3ft)
* PA12	120cm (4ft)
* PA15	150cm (5ft)
* PA18	180cm (6ft)
* PA21	210cm (7ft)
* PA24	240cm (8ft)

permaramp - entry flap™

Internal bridging flap for permaramp adjust.

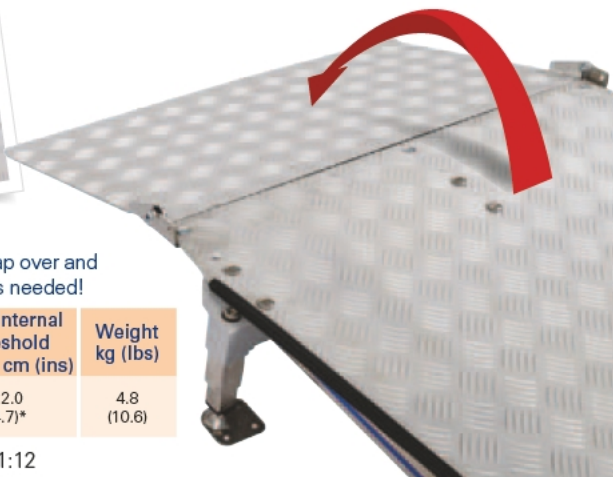

Leave ramp in place, simply folding flap over and down to gain smooth access inside as needed!

Code	Length	Width	Max internal threshold height cm (ins)	Weight kg (lbs)
* PE5	45cm (18ins)	72cm (28.3ins)	12.0 (4.7)*	4.8 (10.6)

* based on external ramp gradient of 1:12

permaramp - super-wide™

- ▶ Suited for bariatric use – up to 400kg (63 stone) capacity
- ▶ Great for scooters, mobility devices, and sports wheelchairs
- ▶ PAW is totally self-supporting and adjustable; great for PVC doors



“Superwide 90cm option; perfect for single outward opening doors”

POW



Code	Length	Capacity kg (st)	Weight kg (st)
* POW9	90cm (3ft)	400 (63)	10 (22)
* POW12	120cm (4ft)	400 (63)	14 (31)
* POW15	150cm (5ft)	400 (63)	18 (40)
* POW18	180cm (6ft)	400 (63)	22 (48)
* POW21	210cm (7ft)	350 (55)	26 (58)
POW24	240cm (8ft)	300 (47)	30 (65)

NEW

Code	Length	Capacity kg (st)	Weight kg (st)
* PAW9	90cm (3ft)	400 (63)	14 (22)
* PAW12	120cm (4ft)	400 (63)	18 (40)
* PAW15	150cm (5ft)	400 (63)	22 (48)
* PAW18	180cm (6ft)	400 (63)	26 (57)
* PAW21	210cm (7ft)	350 (55)	30 (66)
PAW24	240cm (8ft)	300 (47)	34 (75)



PAW

WELCOME[®] by rampCENTRE[®]

ECO FRIENDLY RAMP SYSTEM

Attractive & excellent value, straightforward residential access solution



Fast one person install



Small van friendly



100% Modular



Next day delivery

This modular ramp system has been carefully developed to suit your budgets, needs and timescales. It is incredibly good value, extremely versatile and quick to deliver and install.



Designed & Manufactured in the UK.



CAD drawings available for any variation


Thousands of applications



Accessories available:

Self supporting doorplate

No more cracked PVC door thresholds!

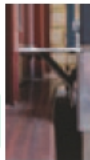

Size cm
80 x 15
100 x 15

Wide/long doorplates

Now available up to 100cm width & 22cm length.



Size cm
80 x 15 100x15
80 x 22 100x22

Platforms



2 sizes of platform available:

Standard platform
120cm (4ft) x 120cm (4ft)

Patio platform
120cm (4ft) x 180cm(6ft)

Platform to platform Joiner

Simply join platforms together.



Steps

Easy access for all users.
(Order 2 kits for 2 steps).



Step handrails



Soft ground leg bearer

Stop legs from sinking in soft ground.



Bowend

For extra safety at the ends of handrails



Leg extensions

Increase max system heights from 33cm to 55cm.




swift ramp system™

Perfect for higher levels and communal or public applications

Your benefits:

- ▲ Rise up to 1.5m (5 feet)
- ▲ Three widths from 90cm (3 feet) to 150cm (5 feet)
- ▲ Suited for bariatric use
- ▲ NEW closed mesh surface – perfect for pets and high heels!

Please call or email to order or for more information

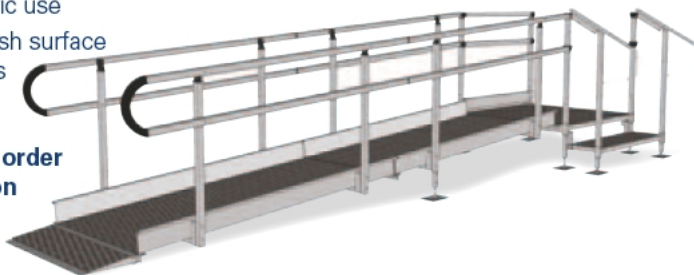

WELCOME[®] by rampCENTRE[®]


ECO FRIENDLY RAMP SYSTEM

“ Substantial half steps with handrails ”

- ▶ 2 huge sizes: XL to fit rollator and L to fit walking frame
- ▶ Handrail and kerb accessories for increased safety



Walking device friendly

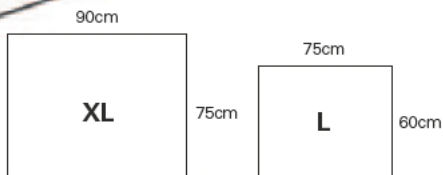

Code	Min Height	Max Height	Weight kg (lbs)
WHSL	7.5cm (3ins)	12.5cm (4.9ins)	10 (22)
WHSXL	7.5cm (3ins)	12.5cm (4.9ins)	12 (26.5)

Half step accessories

RAILS	(Supplied with 2 fix-down feet)
WHS60	60cm (24ins) wide handrail
WHS75	75cm (29.5ins) wide handrail
WHS90	90cm (35.4ins) wide handrail
KERBS	(Use with rails)
WHK60	60cm (24ins) safety kerb
WHK75	75cm (29.5ins) safety kerb
WHK90	90cm (35.4ins) safety kerb
FIX-DOWN FEET	
WDFD	Adjustable fix-down feet (2pk)


TOP TIPS!



1. Refer to Ramp Specifier Guide (p2/3)

2. Always take photographs

3. Make a rough sketch and add a few dimensions

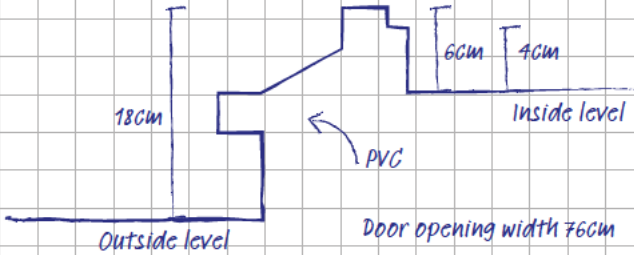

social at another level Inclusion™



Contact us for product selection advice and stockist information



rampCENTRE®

by
Enable Access

Scan below to request
more FREE copies.



easy  **care**
systems

Units 5 to 6 Lower Vicarage Road,
Southampton, Hampshire, SO19 7RJ

T: 02380 420678

E: info@easycareystems.co.uk

W: www.easycareystems.co.uk

