

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


General User/Safety Guide

# **SAFE ROLL ULTRA LOW BUMPER**

SIDERAIL





# CONTENTS

<b>WARNINGS &amp; CAUTIONS</b>	<b>4</b>
<b>PRODUCT OVERVIEW</b>	<b>5</b>
<b>GENERAL INFORMATION</b>	<b>6-7</b>
<b>INSTALLATION</b>	<b>8-9</b>
<b>CLEANING AND CARE</b>	<b>10</b>

# WARNINGS & CAUTIONS



**READ THIS INSTRUCTION MANUAL CAREFULLY. PLEASE NOTE IN PARTICULAR THAT THE SAFETY INSTRUCTIONS MUST BE OBSERVED.**



## WARNING

Please ensure that this booklet is read prior to first use.

Please refer to the information in this booklet as a guideline only.

Harvest Healthcare Ltd recommend that prior to using this product, a full risk assessment must be completed by a qualified professional, in order to determine that the correct product is suitable for each individual.

Do not allow sharp objects to penetrate the Safe Roll Ultra Low bumper or the covering material.

Ensure this product is fitted as specified in the fitting instructions.

Routinely inspected this item for damage and ensure all fixing straps or Hook and Loop are securely fastened.

Remove the item from service if the product is damaged or ripped.

Remove the item if the fixing straps or the Hook and Loop show signs of wear and they are not effective.

Do not expose this product to open flames. We advise the user not smoke when using this product.

Facial contact with the cover may cause suffocation if used incorrectly.

## **SYMBOLS USED**



COVERS ARE  
REMOVABLE AND  
MACHINE WASHABLE



LOW HEAT  
TUMBLE DRY



DO NOT  
DRY CLEAN



DO NOT  
BLEACH



DO NOT IRON



CE MARK



UKCA MARK



DO NOT USE  
PHENOLS



NO  
SMOKING



This symbol indicates general hazards. There is a danger to life and health.

# PRODUCT OVERVIEW

**PRODUCT CODE: HFB002**

## **PRODUCT DESCRIPTION**

### **Safe Roll Ultra Low Bumper for Woburn Ultra-Low Profiling Beds**

The Safe Roll Ultra Low bumper has been designed to fit onto our Woburn Ultra-Low bed. The product fills the need to provide a safe environment for the user who may not need side rails fitting.

The Safe Roll bumper helps to prevent accidental contact with the floor, the material we use is soft to the touch giving the user protection in case they accidentally roll off the bed.

Ideal for nursing homes and care in a domestic environment. The Safe Roll bumper is easy to fit; simply place and fasten the bumper to the side of the bed on which the user is most likely to accidentally roll out. Refer to the fitting instructions (installation section).



# GENERAL INFORMATION

## INSTRUCTIONS FOR USE

Check the product for damage, and if it is the correct type of product for the bed, before carrying out the installation.

- If the Safe Roll Ultra Low bumper or any other part of the product looks damaged do not fit to the bed.
- Refer to the product code and pictures to make sure you're using the right instruction for use.
- Before installing this product read the instruction for use thoroughly and seek advice if you are unsure how to fit the product.

To help prevent injury it is important that patients and equipment are monitored regularly.

The Safe Roll Ultra Low bumper may be delivered in a plastic cover for protection; these covers must be removed and disposed of before the equipment can be used. Please dispose of the plastic as specified by your local waste disposal service authority.

## INTENDED PURPOSE

The Safe Roll Ultra Low bumper provides safety for users at risk of accidentally rolling out of a floor bed when unattended. Only suitable to be used with the Woburn Ultra Low beds. This is an alternative to using a side rail system which might not be suitable for the bed or it could cause injury to the end user if they are prone to involuntary movements.

## SAFE ROLL ULTRA LOW BUMPER SYSTEM INSPECTION RECOMMENDATIONS

The Safe Roll Ultra Low bumper system should be checked regularly to ensure they remain 'fit for purpose', effective and pose no risk of infection to either the patient or the carer.

Harvest Healthcare Ltd recommend that a thorough inspection of both the exterior (cover) zips, straps and Hook and Loop is carried out weekly, or each time a new patient is introduced to this equipment. Visual checks should be carried out daily to identify any significant signs of damage or infection risk.

If any signs of contamination are identified, the side rail system should be withdrawn from use immediately and cleaned.

## CLEANING AND CARE

The bumper system has a protective PU cover. The cover should be cleaned regularly in accordance with your organisation's disinfection protocol.



**Facial contact with any covering material such as that used on this product can cause suffocation and a Risk Assessment should be done prior to the use of this product.**

# GENERAL INFORMATION

## **MOVING, HANDLING AND STORAGE**

Where possible, bumper system should be stacked flat to prevent damage to the product.

It may require more than one person to aid in fitting the system.

The bumper system is to be stored between -5 to +40 degrees, in a dry environment.

Do not expose to high humidity or heat.

**Do not drag or pull the bumper system by the cover.**

## **SAFE USE**

This product is intended as an aid to prevent accidental injury. With a carer present check the material, zips and fastenings to ensure that these parts of the bumper systems have not been damaged and do not introduce gaps or a hazard that could increase the risk of entrapment suffocation or injury to the user.

Daily check the bumper is still fastened to the bed and brakes have been applied on all four castors.

Before and during the use of the bumper, consider the effectiveness of the system. Seek advice to ensure the product is suitable for the bed before attempting to fit it. The fastening straps must loop around the static parts of the bed. Thoroughly read the fitting instructions supplied with this product. (Installation section)

## **Important Notice**

Do not sit on the bumper or allow the bumper to be crushed when the bed is lowered to the floor, as this will deform the product and could result in less protection being provided to the user.

Beds used for Safe Roll Ultra Low bumper installation should always be operated in line with their instructions for use.

## **WARRANTY**

### **Bumper Systems**


All Harvest Healthcare bumper systems are covered by warranty for a period of 1 year.

Damage through incorrect use will invalidate this warranty.

## **GUARANTEE**

Harvest Healthcare Ltd guarantees to repair or replace all goods issued to its customers which are found to be defective whilst still under their applicable warranty period. All warranties are subject to our standard terms and conditions of sale which can be found on our website or by request to Harvest Healthcare Ltd.

# INSTALLATION

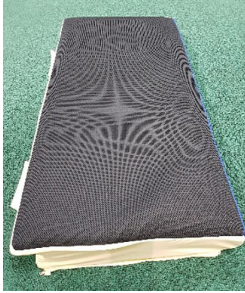

Top surface with spacer mesh material to reduce the risk of suffocation.



Bottom surface to allow bumper to move back into place as bed is lowered.

**NEVER** install with this surface upwards.




Hinge mechanism with Hook and Loop fasteners.



1. Open all the Hook and Loop fasteners.
2. Raise the bed to allow you to position the bumper system under the edge of the sleeping platform as below.

3. Lower the bed carefully to allow the edge of the sleeping platform to locate in the cut out on the edge of the bumper.
4. Close all the Hook and Loop fasteners to secure the bumper in place.
5. Raise the bed to check the system for security.

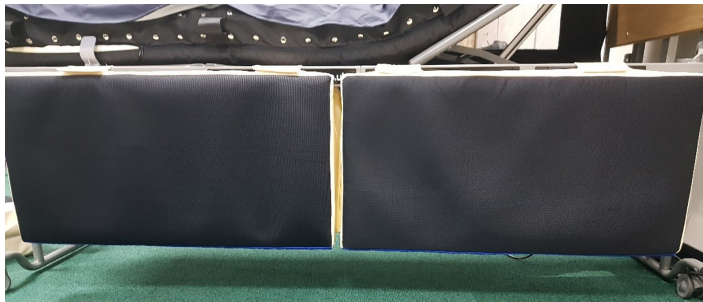



# INSTALLATION

The system should hang securely to the sleeping platform. When the bed is raised completely there should be enough clearance to allow moving and handling equipment access below.

6. Lower the bed and hold the bottom of the bumper outwards from the underside of the bed until it contacts the floor, at this point it should slide on the blue material and settle into place.



7. Once lowered completely the bumper should fit snugly against the bed covering the outer edge of the sleeping platform.



## Please note

- Carry out a risk assessment to ensure it is suitable for the users' requirements and is compatible with the bed.
- Ensure it is installed correctly with the black spacer mesh material upwards.
- Be aware that it could cause a potential trip hazard when giving care.
- Never stand on the bumper system.
- The outer cover can be removed for decontamination.
- Please see the installation guide video on our website.
- In the event of the bumper system becoming damaged discontinue use immediately and contact Harvest Healthcare for advice.

# CLEANING & CARE



## WARNING

Eye protection, gloves and protective clothing should be worn when carrying out cleaning and disinfection procedures.

When disinfecting the system, Harvest Healthcare recommends the following guidelines which have been developed to comply with recognised infection control procedures. These procedures are also to be used to prevent cross infection when transferring the system between patients.

### SAFE ROLL ULTRA LOW BUMPER SYSTEM MATERIAL COVER

During general use the cover can be cleaned by wiping it with a mild detergent solution. Machine wash or wipe the netting with soap and water or a with a sodium hypochlorite solution diluted to 1000ppm and tumble dry at low heat or allow to naturally dry.



Do not use abrasive cleaners, phenol disinfectants, solvents or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.



Do not iron.



Ensure that the side rail netting material is thoroughly dried before refitting the bumper or placing in storage.



**HYPERCARBONATE AND PHENOL-BASED SOLUTIONS SHOULD NOT BE USED AS THESE WILL DAMAGE THE BUMPER COVER**

### DISPOSAL

If contaminated, please dispose of as clinical waste.

If not contaminated dispose of as household waste at your local approved facility.

### Note:

Please report any serious incident related to this device to Harvest Healthcare.

Full contact details can be found at the end of this document.





Special Products and Service

**Contact Us**

Units 5 and 6 Lower Vicarage  
Road  
Southampton  
Hampshire  
SO19 7RJ

T: 02380 420678

E: [info@easycare systems.co.uk](mailto:info@easycare systems.co.uk)

[www.easycare systems.co.uk](http://www.easycare systems.co.uk)