



Product Code: HWB002

Description: Snug Wall Bumper

- The snug wall bumper has been designed to fit onto our Woburn bed range or other similar types of profiling beds produced by various manufactures.
- The product fills the need to provide a safe environment for the user who may not need side rails fitting.
- The bumper helps to prevent accidental contact with the wall, the material we use is soft to the touch giving the end user reassurance and protection.
- Ideal for nursing homes and care in a domestic environment.
- It is easy to fit. Fitting instruction video is available on the website.

Note: After fitting the snug wall bumper check that all the brakes on the bed are applied to prevent any movement of the bed when it is being operated.

Cleaning and Care: The cover is removable and machine washable, for better infection control.
Pre wash cold 10 minutes
Main wash 80°C 10 minutes
Followed by cold rinses and extraction.
Do not use abrasive cleaners, phenol disinfectants, solvents, or alcohol-based cleansers, e.g. Dettol, Phenicol, Hibiscrub, Clearsol, Stericol, Hycoline, as these will damage the cover materials.
Do not iron.
Tumble dry at low heat.

Ensure that the cover is thoroughly dried before inserting the foam and placing the bumper into storage.

Latex free: Yes

User Instruction: Yes

Flammability: BS 7175: 1989 (2013) ignition sources 0,1

Dealer Contact Details:

Easy Care Systems Ltd

Units 5 and 6 Lower Vicarage Road

Southampton

Hampshire

SO19 7RJ

T: 02380 420678

E: info@easycare systems.co.uk