



Instruction for use



www.easycare systems.co.uk

SatinSheet 2Direction
BaseSheet


immedia
by Etac

Index


Illustrations3
User manual – English4




Positioning different sizes of SatinSheet BaseSheet 2Direction on a mattress

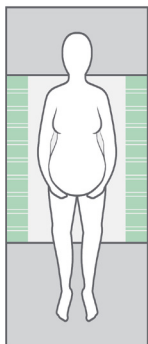


100 x 200 cm

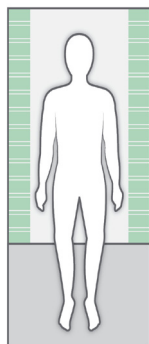


140 x 200 cm

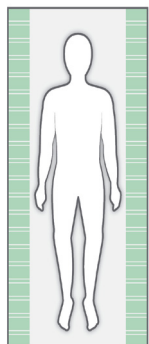

200 x 200 cm

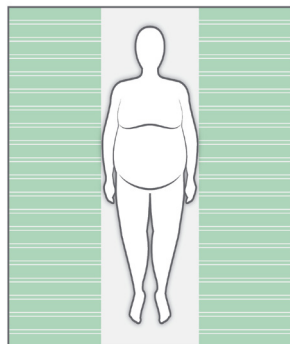


230 x 200 cm

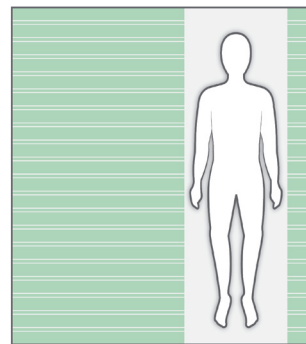


280 x 200 cm


Positioning different sizes of SatinSheet DrawSheet placed correct on top of SatinSheet 2Direction

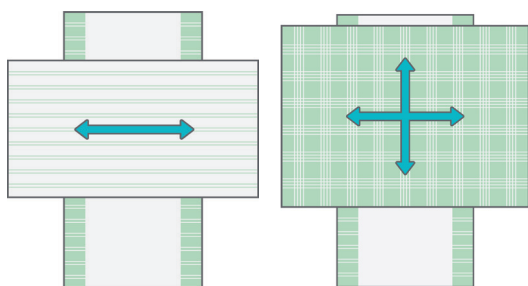


Four-step process for positioning the sheet on a person in a bed.

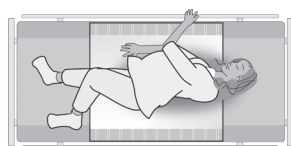

1

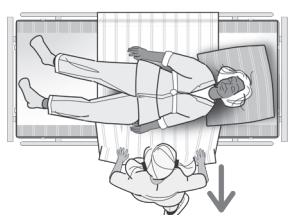

2

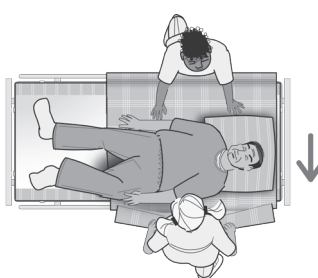

3



4

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, please read this manual carefully.

The person described as “the user” in this “Instruction for Use” is the person who is lying or sitting on the product. The carers are the people who manoeuvre the product.



This symbol appears alongside the text in the manual. It draws the reader’s attention to points at which there may be a risk to the health and safety of the user or carer.

These products comply with the standards applicable for Class 1 products in the Medical Device Regulation (EU) 2017/745 on medical devices.

At Etac we strive to continually improve our products and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including

recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

Expected lifetime: The device has an expected service life of 1 to 2 years under normal use. The service life of the device varies depending on usage frequency, loads and how often and how it is washed.

For further information on the Etac transfer range, see www.etac.com.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Symbols in general

| | | | |
|-----------------|--------------------------|---|----------------|
| Washing | Disinfect / Wipe off | Warning | Manufacturer |
| Do not bleach | Never leave on the floor | Production year, month and day | CE-marking |
| Tumbledry | Risk of sliding down | Batch no./Lot no. | Keep dry |
| Iron | Keep out of the sun | User mass limit = maximum rated load | Non sterile |
| Do not dryclean | Read the user manual | The product can be scrapped according to national regulations | Article number |
| Medical device | | | |

Intended purpose

SatinSheet is a BaseSheet that reduces friction when turning and positioning users in bed and makes work easier for the carer. SatinSheet is positioned in the bed like an ordinary sheet and can be combined with DrawSheets or SatinSheet 4Direction. The SatinSheet range comprises different models

of BaseSheet with low-friction material and DrawSheets. The choice of model or combination is made with reference to the user’s functions and needs and the carers’ working environment.

Intended environment

Acute care, Long-term care, Home care

Positioning

Position SatinSheet with the low-friction material uppermost in the middle of the bed (see Illustrations).

All SatinSheet models have a “this side up” label, to help ensure that the correct side faces upwards in the bed.

An information label, describing the correct use of the product, is sewn into all SatinSheet models/products.

Use

A SatinSheet can be very beneficial where the user is able to turn him/herself but needs to reduce friction. Using SatinSheet the user can turn him/herself without sliding down in the bed, even if the head end is raised (see Illustration 1).

If the user needs help to turn, SatinSheet can be beneficial when used with a draw sheet (Illustration 2).

A SatinSheet 4Direction (IM4115S) can be used to help move the user up and down in the bed and from one side to the other (Illustrations 3+4).

Turning is achieved by grasping the sheet at the user’s hips and shoulders. To achieve a better ergonomic grip, the carer should grasp as much fabric as possible.

Always work with flat wrists and weight transference. Turning with SatinSheet can also be done by two carers. The carers work together. One carer helps pull on DrawSheet while the other carer pulls down on the mattress at the level of the user’s hips and shoulders, pushing gently.

Turn the user (see Illustration 2). To lock the glide system after turning: Fold the excess material from DrawSheet under the mattress on both sides of the bed.

Before locking the glide system under the mattress, any unevenness/creases should be smoothed to avoid decubitus. The carers pull simultaneously and diagonally from both sides.

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques

Encourage the user to assist where possible.

To ensure that the user feels safe and that every manual handling is smooth, always plan the manual handling in advance.



There is a risk that the user may slide off. Never leave the user alone on the edge of the bed.



Never leave the product on the floor.



Always conduct a risk assessment, and ensure that the assistive product can be used for the individual user and, in combination with other devices, that it is safe for the user and carers.



It is recommended that the guard rail is in place when the user is left on the glide system, unless a risk assessment has found that the user can be safely left on the glide system without the need of guard rails.



Always lock the glide system when you have completed the transfer.

If in any doubt - please contact Etac.

Material and cleaning

Material:

Polycotton: 55% cotton/45% polyester.

Satin: Polyester

The product should be checked regularly, preferably each time it is used and especially after washing.

Check that there is no damage to seams or fabric.



**Never try to repair a product yourself.
Never use a defective product.**



Recommended to wash the product before use.

Washing instructions:


Ironing improves the glide effect.

Do not use fabric softener.


This product can be cleaned with a solvent-free cleaning agent with a pH level between 5 and 9, or a 70% disinfection solution.




| Article no. | Product Description | Size (mm) |
|-------------|---|-------------|
| IM4133S | SatinSheet 2Direction mini - small (satin W600xL1050 mm) | W1950xL1050 |
| IM4108S | SatinSheet 2Direction mini (satin W700xL1050 mm) | W1950xL1050 |
| IM4175S | SatinSheet 2Direction mini - wide (satin W850xL1050 mm) | W2300xL1050 |
| IM4178S | SatinSheet 2Direction mini - double (satin W960xL1050 mm) | W2300xL1050 |
| IM4119S | SatinSheet 2Direction midi - small (satin W600xL1400 mm) | W2000xL1400 |
| IM4113S | SatinSheet 2Direction midi (satin W700xL1400 mm) | W2000xL1400 |
| IM4176S | SatinSheet 2Direction midi - wide (satin W850xL1400 mm) | W2300xL1400 |
| IM4179S | SatinSheet 2Direction midi - double (satin W960xL1400 mm) | W2300xL1400 |
| IM4181S | SatinSheet 2Direction maxi - small (satin W600xL2000 mm) | W2000xL2000 |
| IM4114S | SatinSheet 2Direction maxi (satin W700xL2000 mm) | W2000xL2000 |
| IM4177S | SatinSheet 2Direction maxi - wide (satin W850xL2000 mm) | W2300xL2000 |
| IM4180S | SatinSheet 2Direction maxi - double (satin W960xL2000 mm) | W2300xL2000 |
| IM4122S | SatinSheet 2Direction 180 cm bed (satin W600xL2000 mm) | W2800xL2000 |
| IM4167S | SatinSheet 2Direction Corner midi - small (satin W800xL1400 mm) | W900xL1730 |
| IM4126S | SatinSheet 2Direction Corner mini (satin W650xL1050 mm) | W850xL2000 |
| IM4157S | SatinSheet 2Direction Corner midi (satin W650xL1400 mm) | W850xL2000 |
| IM4158S | SatinSheet 2Direction Corner midi - wide (satin W800xL1400 mm) | W1050xL2000 |
| IM4187S | SatinSheet 2Direction Corner midi - double (satin W900xL1400 mm) | W1200xL2000 |
| IM4107S | SatinSheet 2Direction Corner (satin W650xL2000 mm) | W850xL2000 |
| IM4147S | SatinSheet 2Direction Corner wide (satin W800xL2000 mm) | W1050xL2000 |
| IM4137S | SatinSheet 2Direction Corner double (satin W900xL2000 mm) | W1200xL2000 |
| IM4197S | SatinSheet 2Direction Corner maxi - double (satin W1550xL2000 mm) | W1800xL2000 |
| IM4112S | SatinSheet 2Direction Fit (satin W700xL2000 mm) | W900xL2000 |
| IM4142S | SatinSheet 2Direction Elastic Fit (satin W700xL2000 mm) | W900xL2000 |

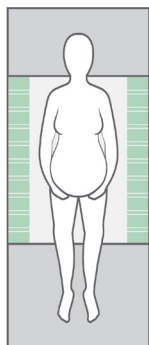

| Article no. | BaseSheet - Mini |
|---|--|
| IM4133S | SatinSheet 2Direction mini - small |
| IM4108S | SatinSheet 2Direction mini |
| IM4175S | SatinSheet 2Direction mini - wide |
| IM4178S | Satinsheet 2Direction mini - double |
| IM4126S | SatinSheet 2Direction Corner - mini |
| BaseSheet - Mini can be combined with: | |
| Article no. | DrawSheet and 4Direction |
| IM4116S | SatinSheet Drawsheet - mini |
| IM4110S | In2Sheet 4Direction w/wings - mini |
| IM4110SH | In2Sheet 4Direction w/wings + handles - mini |

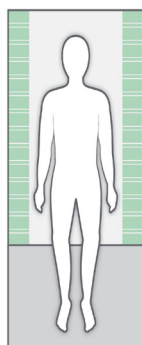

| Article no. | BaseSheet - Midi |
|---|--|
| IM4119S | SatinSheet 2Direction midi - small |
| IM4113S | SatinSheet 2Direction midi |
| IM4176S | SatinSheet 2Direction midi - wide |
| IM4179S | SatinSheet 2Direction midi - double |
| IM4167S | SatinSheet 2Direction Corner - small - midi |
| IM4157S | SatinSheet 2Direction Corner - midi satin |
| IM4158S | Satinsheet 2Direction Corner midi-wide |
| IM4187S | Satinsheet 2Direction Corner midi-double |
| IM4107S | SatinSheet 2Direction Corner |
| BaseSheet - Midi can be combined with: | |
| Article no. | DrawSheet and 4Direction and In2Sheet 4Direction |
| M4118S | SatinSheet Drawsheet - midi |
| IM4115S | SatinSheet 4Direction - midi |
| IM4115SH | SatinSheet 4Direction w/handles - midi |
| IM4140S | In2Sheet 4Direction w/wings - midi |
| IM4140SH | In2Sheet 4Direction w/wings + handles - midi |

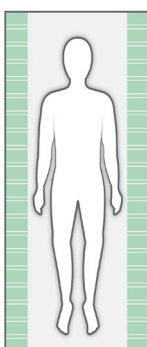

| Article no. | BaseSheet - Maxi |
|--|--|
| IM4181S | SatinSheet 2Direction maxi - small |
| IM4114S | SatinSheet 2Direction maxi |
| IM4177S | SatinSheet 2Direction maxi - wide |
| IM4180S | SatinSheet 2Direction maxi - double |
| IM4122S | SatinSheet 2Direction 180 cm bed |
| Article no. | BaseSheet - Maxi Corner |
| IM4107S | SatinSheet 2Direction Corner |
| IM4147S | SatinSheet 2Direction Corner - wide |
| IM4137S | SatinSheet 2Direction Corner - double |
| IM4197S | SatinSheet 2Direction Corner - double - maxi |
| Article no. | BaseSheet - Maxi Fit |
| IM4112S | SatinSheet 2Direction Fit |
| IM4142S | SatinSheet 2Direction Elastic Fit |
| BaseSheet - Maxi/Corner/Fit can be combined with: | |
| Article no. | 4Direction and In2Sheet 4Direction |
| M4115S | SatinSheet 4Direction - midi |
| IM4115SH | SatinSheet 4Direction w/handles - midi |
| IM4140S | In2Sheet 4Direction w/wings - midi |
| IM4140SH | In2Sheet 4Direction - w/wings + handles - midi |
| IM4121S | SatinSheet 4Direction - maxi |
| IM4121SH | SatinSheet 4Direction w/handles - maxi |

Remember SatinSheet PillowCase (2D)

| | |
|---------|----------------------------------|
| IM4117S | SatinSheet 2Direction PillowCase |
|---------|----------------------------------|

| Article no. | Description | SatinSheet 2Direction BaseSheet | SatinSheet DrawSheet and SatinSheet 4Direction |
|--------------|--|---|--|
| IM4201S | SatinSheet DrawSheet mini set | IM4108S SatinSheet 2Direction mini | IM4116S SatinSheet DrawSheet - mini |
| IM4210S | SatinSheet In2Sheet mini set | IM4108S SatinSheet 2Direction mini | IM4110S In2Sheet 4Directions w/wings - mini |
| IM4209S | SatinSheet In2Sheet midi mini set | IM4119S SatinSheet 2Direction midi - small | IM4110S In2Sheet 4Directions w/wings - mini |
| IM4205S | SatinSheet DrawSheet midi small set | IM4119S SatinSheet 2Direction midi - small | IM4118S SatinSheet DrawSheet - midi |
| IM4200S | SatinSheet DrawSheet midi set | IM4113S SatinSheet 2Direction midi | IM4118S SatinSheet DrawSheet - midi |
| IM4207S | SatinSheet 4Direction midi small set | IM4119S SatinSheet 2Direction midi - small | IM4115S SatinSheet 4Direction - midi |
| IM4204S | SatinSheet 4Direction midi set | IM4113S SatinSheet 2Direction midi | IM4115S SatinSheet 4Direction - midi |
| IM4202S | SatinSheet DrawSheet maxi midi set | IM4114S SatinSheet 2Direction maxi | IM4118S SatinSheet DrawSheet - midi |
| IM4203S | SatinSheet 4Direction Fit midi set | IM4112S SatinSheet 2Direction Fit | IM4115S SatinSheet 4Direction - midi |
| IM4208S | SatinSheet Corner DrawSheet midi set | IM4107S SatinSheet 2Direction Corner | IM4118S SatinSheet DrawSheet - midi |
| IM4211S | SatinSheet Corner 4Direction midi set | IM4107S SatinSheet 2Direction Corner | IM4115S SatinSheet 4Direction - midi |
| IM4212S | SatinSheet Corner 4Direction maxi set | IM4107S SatinSheet 2Direction Corner | IM4121S SatinSheet 4Direction - maxi |
| IM4213S | SatinSheet 4Direction maxi set | IM4114S SatinSheet 2Direction maxi | IM4121S SatinSheet 4Direction - maxi |
| IM4214S | SatinSheet 4Direction maxi set - wide | IM4177S SatinSheet 2Direction maxi - wide | IM4121S SatinSheet 4Direction - maxi |
| IM4215S | SatinSheet 4Direction maxi set - double | IM4180S SatinSheet 2Direction maxi - double | IM4121S SatinSheet 4Direction - maxi |
| IM4216S | SatinSheet 4Direction midi set - wide | IM4176S SatinSheet 2Direction midi - wide | IM4115S SatinSheet 4Direction - midi |
| IM4217S | SatinSheet 4Direction midi set - double | IM4179S SatinSheet 2Direction midi - double | IM4115S SatinSheet 4Direction - midi |
| IM4218S | SatinSheet 4Direction maxi midi set | IM4114S SatinSheet 2Direction maxi | IM4115S SatinSheet 4Direction - midi |
| IM4219S | SatinSheet 4Direction maxi midi set - wide | IM4177S SatinSheet 2Direction maxi - wide | IM4115S SatinSheet 4Direction - midi |
| IM4220S | SatinSheet 4Direction maxi midi set - double | IM4180S SatinSheet 2Direction maxi - double | IM4115S SatinSheet 4Direction - midi |
| IM4221S | SatinSheet DrawSheet maxi midi set - wide | IM4177S SatinSheet 2Direction maxi - wide | IM4118S SatinSheet DrawSheet - midi |
| IM4222S | SatinSheet DrawSheet maxi midi set - double | IM4180S SatinSheet 2Direction maxi - double | IM4118S SatinSheet DrawSheet - midi |
| IM4223S | SatinSheet DrawSheet midi set - wide | IM4176S SatinSheet 2Direction midi - wide | IM4118S SatinSheet DrawSheet - midi |
| IM4224S | SatinSheet DrawSheet midi set - double | IM4179S SatinSheet 2Direction midi - double | IM4118S SatinSheet DrawSheet - midi |
| IM4226S | SatinSheet Corner 4Direction midi set - double | IM4137S SatinSheet 2Direction Corner - double | IM4115S SatinSheet 4Direction - midi |
| IM4227S | SatinSheet Corner DrawSheet midi set - wide | IM4147S SatinSheet 2Direction Corner - wide | IM4118S SatinSheet DrawSheet - midi |
| IM4228S | SatinSheet Corner DrawSheet midi set - double | IM4137S SatinSheet 2Direction Corner - double | IM4118S SatinSheet DrawSheet - midi |
| IM4229S | SatinSheet Corner 4Direction maxi set - wide | IM4147S SatinSheet 2Direction Corner - wide | IM4121S SatinSheet 4Direction - maxi |
| IM4230S | SatinSheet Corner 4Direction maxi set - double | IM4137S SatinSheet 2Direction Corner - double | IM4121S SatinSheet 4Direction - maxi |
| IM4231S-RCNL | SatinSheet Corner 4Direction w/handle midi - set with dutch insert | IM4107S SatinSheet 2Direction Corner | IM4115SH SatinSheet 4Direction w/ handles - midi |
| IM4232S | SatinSheet 4Direction w/handles midi small set | IM4119S SatinSheet 2Direction midi - small | IM4115SH SatinSheet 4Direction w/ handles - midi |
| IM4233S | SatinSheet 4Direction midi set - wide | IM4176S SatinSheet 2Direction midi - wide | IM4115SH SatinSheet 4Direction w/ handles - midi |
| IM4234S | SatinSheet 4Direction w/handles maxi set - wide | IM4177S SatinSheet 2Direction maxi - wide | IM4121SH SatinSheet 4Direction w/ handles - maxi |
| IM4235S | SatinSheet 4Direction w/handles midi set - double | IM4179S SatinSheet 2Direction midi - double | IM4115SH SatinSheet 4Direction w/ handles - midi |
| IM4236S | SatinSheet 4Direction w/handles maxi set - double | IM4180S SatinSheet 2Direction maxi - double | IM4121SH SatinSheet 4Direction w/ handles - maxi |
| IM4238S | SatinSheet 4Direction maxi set - small | IM4181S SatinSheet 2Direction maxi - small | IM4121S SatinSheet 4Direction - maxi |
| IM4239S | SatinSheet 4Direction w/handles maxi set - small | IM4181S SatinSheet 2Direction maxi - small | IM4121SH SatinSheet 4Direction w/ handles - maxi |
| IM4242S | SatinSheet DrawSheet mini set - wide | IM4175S SatinSheet 2Direction mini - wide | IM4116S SatinSheet DrawSheet - mini |
| IM4243S | SatinSheet DrawSheet mini set - double | IM4178S SatinSheet 2Direction mini - double | IM4116S SatinSheet DrawSheet - mini |
| IM4249S | SatinSheet In2Sheet midi midi set | IM4119S SatinSheet 2Direction midi - small | IM4140S In2Sheet 4Direction w. wings IM4245S SatinSheet Corner DrawSheet midi set IM4208S SatinSheet Corner DrawSheet midi set |

Inspection

First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from www.etac.com or contact your Etac Customer Service or the local dealer.
- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

Periodic Inspection:

- Make sure that your Manual transfer system always is in a perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.




Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.



Stability test:

Try to bend the product verify that it feels firm, stable and solid.

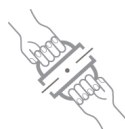

Mechanical load or stress/Test handles:

Draw hard in the handles in opposite directions and control the material, seams and stitching.

Test buckles:

Lock the buckle and draw in opposite directions, control the buckle and the stitching.



Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.

Low friction – slides effortlessly
High friction – no sliding/moving

Check:

Manual transfer aids with handles, straps and buckles:



Manual transfer aids with for sitting and lying transfer and with high or low friction:



Production year, month and day



The product can be scrapped in accordance with national regulations



Read the user manual



Special Products and Service

Contact Us

Units 5 and 6 Lower Vicarage Road Southampton
Hampshire
SO19 7RJ
T: 02380 420678
E: info@easycare systems.co.uk