

KINGFISHER™

Bath/Shower Board

How to assemble, fit and general maintenance



easycare
systems




The Helping Hand
Company





How to Assemble

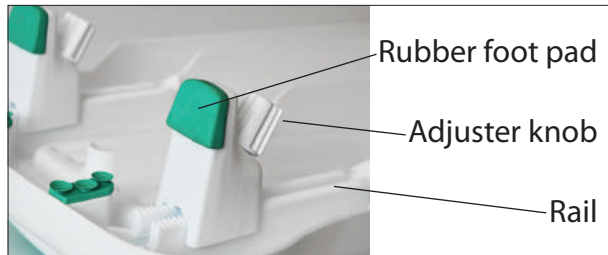

The bath board is width adjustable to fit securely against side of different width/shape baths



With the rubber foot pad facing outwards, slide the 'tongue' under the rail (pull gently up to ensure the foot is engaged under the rail as it should not lift off)



Loosen the adjuster knob on each of the feet (x4) to make it easier to slide on to the bath board



Slide the foot along the rails and repeat so all four feet are correctly in place. Do not tighten up just yet as you will do this when you fit the bath board onto the bath

How to Attach the Bath Board Handle



The optional bath board handle comes with three parts – a handle and two fixing plugs



Before fitting, make sure the handle inclines inwards along the board. This ensures there is space for you to hold onto the handle



Right side up, slot the handle in. The handle will only fit at one end of the board (same end as soap dish)




Turn the bath board over and tap the plugs in to each hole where the handle came through. No tools are required. The handle is now fitted

How to Fit



Place the board over the top of the bath, opposite end to the taps. Make sure the soap dish (and handle, if supplied) are closest to the wall. Push down firmly so the suckers grip the bath rim

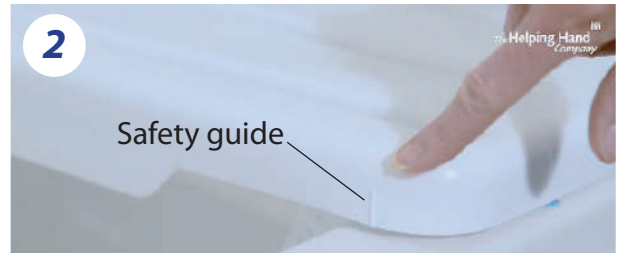

Before going any further, do a visual check that the internal bath width does not exceed the safety guide mark on both ends of the board before continuing



From above, with one hand, slide one foot out to make firm contact with the side of the bath. With the other hand, tighten the adjuster knob underneath



Repeat with all four feet. Try moving the board side to side - it should be fixed firmly in place. The bath board is now ready to use




Max. Safe User Weight 200kg (31 stone)

CHECK, EVERY TIME BEFORE USE, THE BOARD IS FIRMLY IN PLACE

It is not recommended that the bath board be fitted where there is an overlap of more than 25mm




For safety reasons never sit on the edge of the bath board if it overlaps the bath

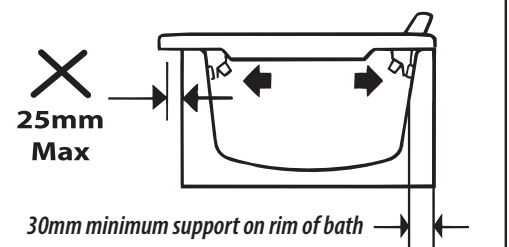


The bath/shower board will not fit a corner bath

Please take care getting in and out of the bath

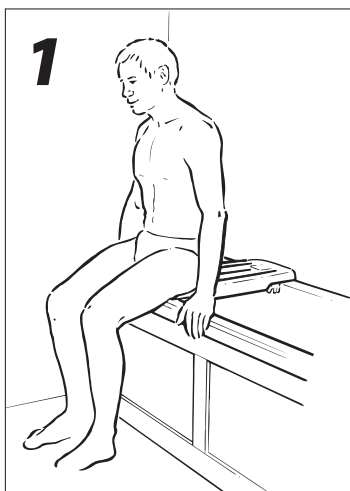

Sit on the end of the board with your hands either side

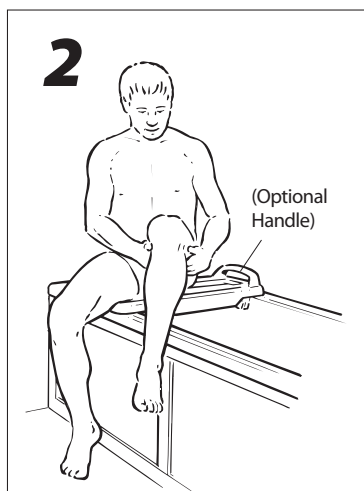

Lift each leg - one at a time - into the bath



Then slide across to the middle of the board

Taking care of the **KINGFISHER™ Bath Board**

1 a) We recommend to keep clean **between uses**, using non-abrasive disinfectant suitable for plastic baths, not forgetting the feet

b) **Before re-issue**, clean and disinfect product using standard detergents and disinfectant wipes

Also compatible for automatic cleaning, using fluid with pH 5-8, up to 105°C for 10 minutes

2 Always inspect the board for wear and tear and **secure fitting before each use**

3 Three year warranty, spare parts available

This device is intended to be used by the following patient groups:

- **Elderly or less mobile**
 - No longer able to safely climb in/out of bath
 - Reduced mobility or reduced strength to transfer safely
 - Ability to sit unsupported/unaided
- **Not suitable for**
 - Users unable to sit without support
 - Children

Disposal/End of use

 **Product is 100% recyclable**

- Community equipment premises - product must be cleaned and disinfected prior to disposal within the local facilities waste management guidelines
- Home use - product should be cleaned and disinfected prior to disposal in domestic waste

Storage

- DO NOT store in sunlight
- Operating temperature +10°C / +40°C
- Storage temperatures -10°C / +40°C



easy**care**
systems




The **Helping Hand**
Company

Units 5 and 6 Lower Vicarage Road
Southampton
Hampshire
SO19 7RJ
T: 02380 420678
E: info@easycare systems.co.uk W:
www.easycare systems.co.uk