



Your guide to Repose Lite Cushion



repose[®]



Why have I been given this product?

You have been given this product by your nurse or healthcare provider because you have been assessed as being at increased risk of developing a pressure ulcer (also known as a 'bed sore' or 'pressure sore'). The Repose range of products are highly effective at redistributing pressure and preventing damage to your skin.

What is a pressure ulcer?

A pressure ulcer is a localised injury to the skin and/ or underlying tissue, usually over a bony area, as a result of pressure.

How does Repose work?

Repose is made from a flexible, multi-stretch material which gently conforms to your shape, surrounding and supporting the body by redistributing pressure across a wide surface area thus protecting vulnerable parts of the skin surface and the tissue below.

How do I install my product?

1. Remove all packaging.
2. Position the cushion onto the chair as far back as possible, with the strap to the rear.
3. Align the centre line of the cushion to the centre line of the chair, ensuring the cushion is not twisted or trapped under the armrests.
4. Feed the strap and clasp around the backrest of the chair (FIG 1).
NOTE: Where possible the straps should be used to secure the cushion in position.
5. Pass the strap through the clasp and tension by pulling through the clasp until you are satisfied that the cushion is held securely in place (FIG 2).
6. Close the clasp (FIG 3).



Fig 1

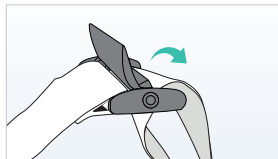


Fig 2

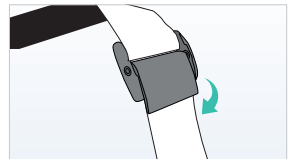


Fig 3

How do I clean my product?

1. Clean and decontaminate as per the instruction for use.



Repose products are
Multi-Patient Use

Repose Lite Cushion Cover Available

A Repose Blue Cushion Cover is also available for the Repose Lite Cushion offering a unique level of protection and patient comfort.

For further information please visit
www.easycare systems.co.uk



Repose
Blue
Cushion
Cover

Repose® Lite Cushion Inspection Guide

During routine use, the appearance of Repose Lite Cushion may change, this is normal and to be expected.

The cushion is supplied with a quick and easy check, allowing users to have peace of mind that their cushion is working as intended.

Conduct this check routinely and between patient use.

1. Place the cushion on a flat surface, undo the strap and feed over the centre of the cushion as shown (FIG 4).
2. Position the end of the strap level with the edge of the cushion.



Fig 4

SUITABLE FOR USE

If the guide mark is visible to the left of the cushion, then it is suitable for use (FIG 4).



Fig 5

NOT SUITABLE FOR USE

If the guide mark is visible over the cushion, then it is not suitable for use (FIG 5).



Please make sure you look after your Repose product and it will look after you.



Keep away from sharp objects and pets



Keep away from direct heat (including cigarettes)



Repose products are Multi-Patient Use



Returning your product

When your Repose product is no longer required, please ensure it is returned to your healthcare provider.





Special Products and Service

For more information:

**Units 5 and 6 Lower Vicarage
Road**

**Southampton
Hampshire
SO19 7RJ**

T: 02380 420678

E: info@easycare systems.co.uk

repose 