



Instructions For Use

www.easycare systems.co.uk

SafeSide

Your SafeSides have been fully tested before leaving the factory. After removing SafeSides from the box discard any tape that has been used for packaging.



Instructions For Use

- Inflate all the tubes using the pump supplied. Ensure that all tubes are adequately inflated, i.e. sufficiently rigid to give a 'ping' when flicked with the finger, but take care not to over inflate the tubes. The stoppers must be pushed home fully to ensure no leakage of air.
- Turn SafeSides upside down. Open mattress compartment and wrap the mattress securely in the underside envelop using the long hook and loop straps. Then invert SafeSides together with the mattress onto the bed base / mattress support.
- Use lateral straps on the underside of the envelope to secure SafeSide onto the bed base / mattress support to give added stability.
- If SafeSides are to be used on a nursing bed or a bed with head and foot boards, it may be necessary to remove head / foot board.
- The washable quilted sheet provided should be replaced on the blue base sheet for added comfort.
- The fold-down sides are secured with the hook and loop straps - when in use ensure the straps are firmly closed.
- Occasionally the tubes may need topping up with air and should be checked regularly.

The Pump

Use two hands when using the pump. The pump is dual action and works on both the up and down strokes. The upper outlet on the pump is used to inflate SafeSides, the lower outlet is used to suck out all the air prior to packing away.

Care and Cleaning

SafeSides will not profile.

Ensure the fold down sides do not rub on the floor when lowered as this may cause the laminate to be worn away through abrasion.

Do not stand or kneel on the fold down side.

The SafeSides can be sponged clean with warm soapy water.

The bed protection pads may be machine washed 5°C and tumble dry moderate heat.

- **DO NOT use abrasive or solvent based cleaners.**
- **DO NOT immerse in water**
- **DO NOT dry Clean**
- **DO NOT ron**

Please note:

As with all bed surrounds, a competent person should undertake a risk assessment as set out in MDA Device Bulletin No. 2001(04) The Safe Use of Bed Rails.

Guarantee

SafeSides are manufactured from an extremely tough polyurethane impregnated nylon to give long lasting performance and an easy clean surface. This shiny surface cannot be repaired by the use of adhesives.

Any attempted repairs will invalidate the Guarantee. In case of puncture please contact Frontier Medical Group for assistance SafeSides are guaranteed for 12 months against faulty workmanship and materials. Repairs will be charged for if, in our opinion, damaged has been caused by misuse.



Special Products and Service

Contact Us

Units 5 and 6 Lower Vicarage Road

Southampton

Hampshire

SO19 7RJ

T: 02380 420678

E: info@easycareystems.co.uk

