





Easy to use sock and stocking aid. Ideal for people struggling to bend, limited mobility, back pain, recovering from surgery.

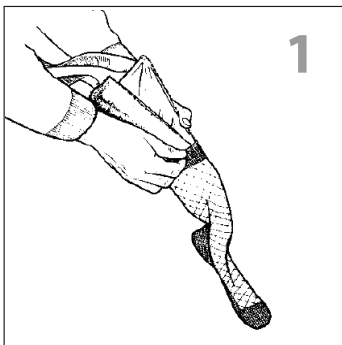

Slide your sock or stocking onto the frame. Pull the sock up to the top of the frame, leaving the opening clear.

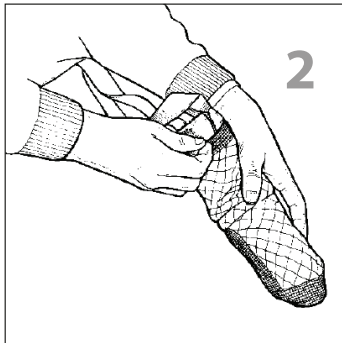

Place the Soxon on the floor while holding the pull loops in your hands/around wrists. Slide foot in and work toes to the end of the sock. Pull up steadily as required. For single arm use, hold pull loops together in one hand.

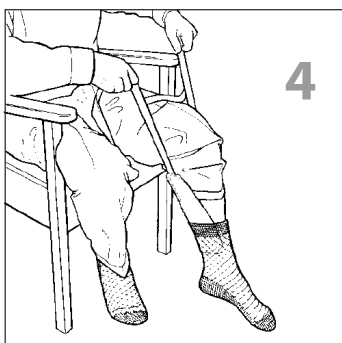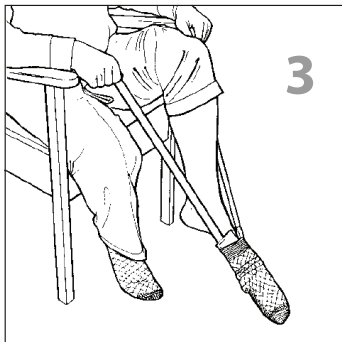



- GB The Soxon DOES NOT work with compression or TED support stockings.
DO NOT use if you feel any discomfort in your back or hip and consult a health professional.
To prevent cross infection the Soxon should only be used by one person.
To Clean - machine wash at 40° or hand wash, using mild detergent.
Disposal/End of use - Product should be cleaned and disinfected prior to disposal in domestic waste.
Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

easyCare
systems

For more information:

Units 5 and 6 Lower Vicarage Road
Southampton
Hampshire
SO19 7RJ
T: 02380 420678
E: info@easycare systems.co.uk W:
www.easycare systems.co.uk

 Product is recyclable 12 month warranty



