

Instruction Manual

www.easycaresystems.co.uk



# The External - Instruction Manual easy easy easy

### Instructions

- 1. Mark out fixing points on a suitable surface
- 2. The height from ground should match the user's requirements
- 3. Drill holes to suit the wall fixings (Not supplied)
- 4. Attach to the surface securely using the 4 fixing positions
- 5. Ensure that the product is safe to use.

### General safety warning:

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference. The product must be installed by a competent professional onto a surface that is deemed suitable.


Should you see any damage during unpacking or suspect a fault, do not install the product and contact your supplier immediately.

Products have been tested to a load of 160kg for 60 seconds in accordance with BS ISO 17966:2016 on a suitable surface.


## Disclaimers, Installation and further safety information:

- 1. This product must be correctly assembled, installed, and maintained.
- 2. Support rails can be slippery when wet use this product with caution.
- 3. It is recommended that this product is installed by a suitably qualified, competent person.
- 4. Only install the product in an area where the mounting surface is structurally sound. (Strengthening modifications to the fixing surface may be required.)
- 5. This product must be secured to the wall using fasteners which are appropriate to the structural base material.
- 6. Any screw caps that are used must be securely fixed so that they cannot be removed during normal usage.


#### Simplified product dimensions

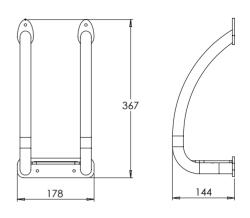


Wall fixing locations

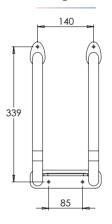

Plant pot hole size

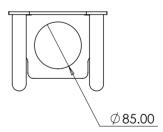

#### Components that can be recycled

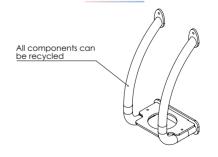



info@easycaresystems.co.uk



## Get in touch

easycare

info@easycaresystems.co.uk 02380 420 678 | easycaresystems.co.uk